

Exchanging a Tafonius PC Monitor

- Make sure the PC is turned OFF
- Remove the lower panel/cover at the back of the PC to reveal the USB and Power cables
- Take a picture of the Power and USB cable positions
- Remove all the cable connections to the back of the PC
- Undo and remove the two lower screws of the VESA hanging bracket that is screwed to the back of the PC
- Partly undo the two upper screws of the VESA hanging bracket
- The PC can then be lifted up and out of the slots of the bracket.
- Re-attach the lower panel/cover to the back of the PC (it may have the PC's backup USB stick attached).
- Take the two top hanging screws out of your old PC and put them in the new PC and then hang the new PC
- Fit the two lower VESA bracket screws and then tighten all 4 VESA bracket mounting screws
- Remove the lower back panel from the new PC
- Attach the Power, USB and data cables to the new PC in the same positions as the old PC
- Feed the cables through the cable-tidy inside the lower panel area
- Fit the lower panel back on the PC
- The red light on the bottom front right of the PC should now be ON.
- If it is not, check all the power and cable connections
- When the red light is showing, press the power button to start the PC
- Fit the old PC in the Monitor box in the same manner that it was sent to you.
- Package securely and return to Vetronic Services.

