



Fitting a new Air or Oxygen Valve to your Tafonius Flow meter

Important


The supplied valve has an attached knob that is held on with two small grub screws. This knob has been accurately aligned on the spindle and should **NOT** be removed.

Removal of old valve control

Disconnect the oxygen or Air supply

Open the valve fully to expose the shaft of the valve

Use a 9/16 spanner on the shaft to turn the valve unit anti-clockwise and unscrew the valve unit.

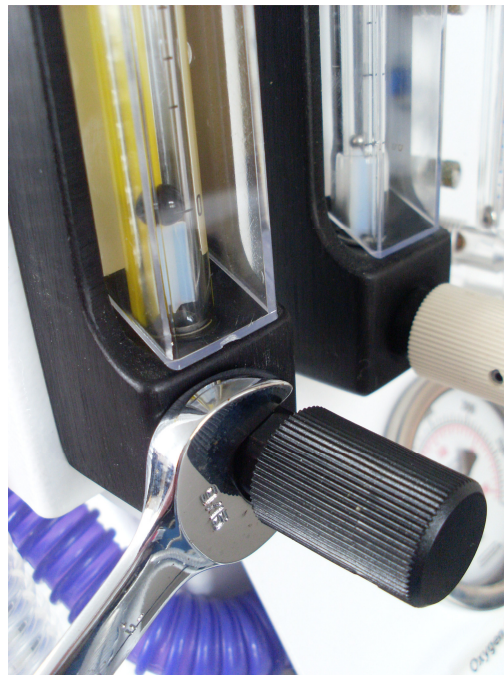

Fitting the new valve control

Make sure the control knob is fully open (anti-clockwise)

Screw in the new valve control until it is finger-tight.

Then tighten it a further 1/8 turn using the spanner.

Now close the control knob in the normal way

Turn on the oxygen or air supply.

Return the old valve control to your supplier

END