



New adaptors for Tafonius 2" tubes

July 2019

We are pleased to announce an enhancement to the 2" patient tubing system used on all Tafonius machines.

The new adaptors can be used with both old and new tubes and will improve the seal in the patient circuit.



The 2" tubes can stretch with time and become loose or can start to leak at the joins with the Y-piece or the valve block.

Using the new adaptors, old tubes can be given a new lease of life. They can also be used on new tubes to improve ease of connection and to prevent leaks.

The adaptors consist of a metal inner former and an outer rubber coupler and the parts are easy to fit with no special tools.

The metal former is pressed into the ends of the 2" patient tube as far as it will go until the tube meets the small retaining lip. The rubber coupler is then pushed over the top of the tube and pushed as far down as possible to form the new coupling.

When tubes become damaged or need to be replaced, the coupler and former can be removed and used on the new tubes.

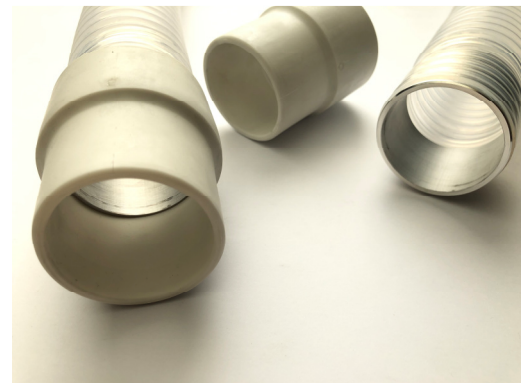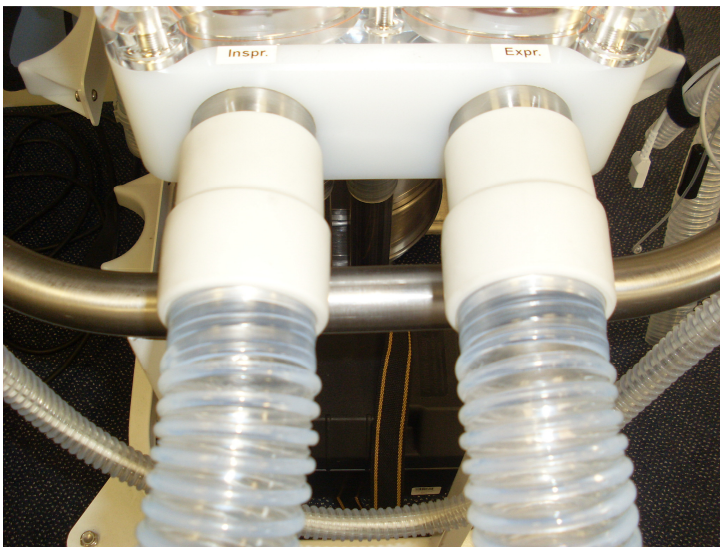

Ordering & Pricing:

VS0430:	2" Former	£12.50	each
200A5884	2" Coupler	£ 15.00	each

VS0471	Single tube kit	£55.00
VS0470	Dual tube kit	£100.00

All prices exclude VAT & Shipping