


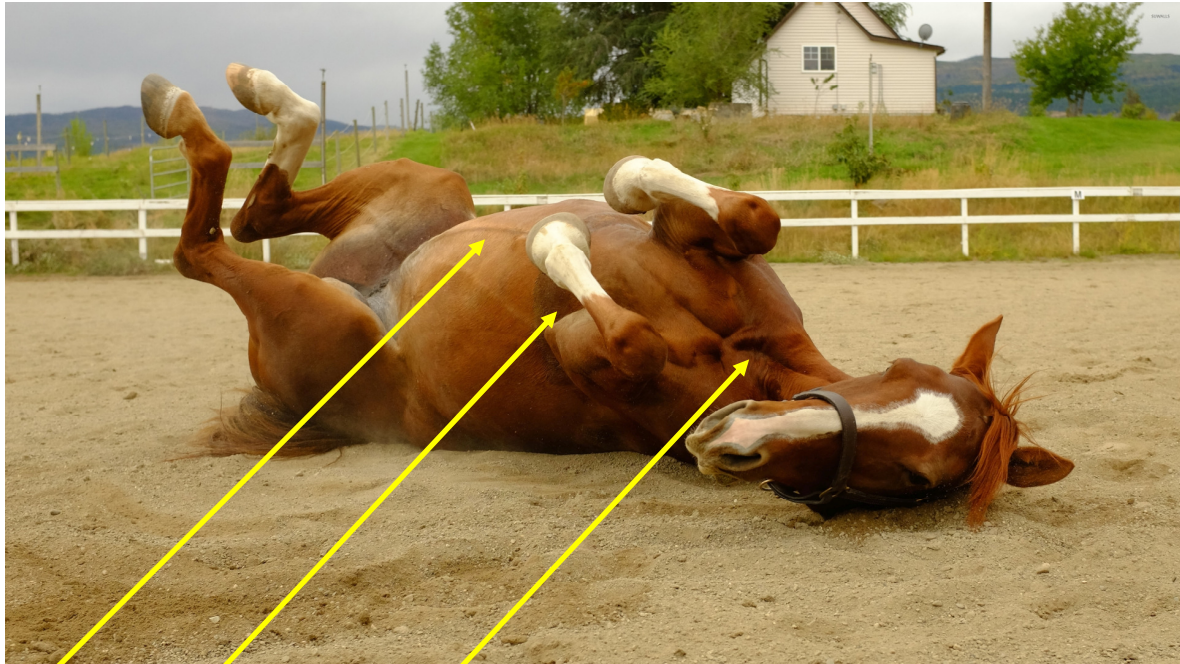

Left Hind
(+ve)

Right Hind
(ref)

Right Fore
(-ve)

UK standard colours:

Right Fore - Red
Left Fore - Yellow
Left Hind - Green
Right Hind - Black

US Colours:

Right Fore - White
Left Fore - Black
Left Hind - Red
Right Hind - Green

Method:

- Put Right Fore Lead in Jugular furrow
- Put Left Hind Lead on sternum near xiphoid process
- Put black somewhere else (reference voltage only)

This will produce a trace in the Lead II channel of the ECG monitor.
The ECG trace will appear to be upside down compared to a standard ECG from a small animal. Reversing the positions of the R Fore/ L Hind leads will reverse the polarity of the signal