


Connecting a Tafonius PC Monitor to your network

Your Tafonius PC Monitor has a Local Area Network (LAN) connection that will allow you to connect it to the internet to allow remote access from technical support for this product. There are two types of PC Monitor, the earlier 1529L model and the 15A1 model. The 1529L model only has one RJ connector port, whereas the 15A1 has 3, only one of which is a LAN connection.



1529L Connector


15A1 Connector

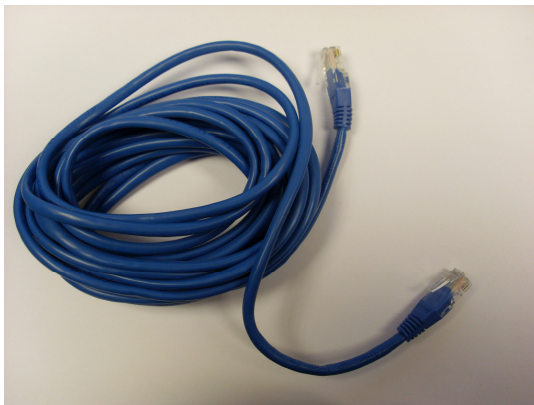


Connect one end of your LAN cable to your internet wall connection and the other to the Tafonius PC. You can do this with the PC turned ON or OFF.

A typical LAN cable looks like the cable on the left


When the PC is running you will need to run a program called TeamViewer. The Teamviewer logo is situated on the Desktop and will look like the following picture, although the number may not be 4.

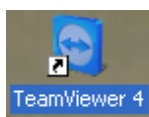

Run Teamviewer by double clicking on the icon, or you can find the program through the Start Menu:

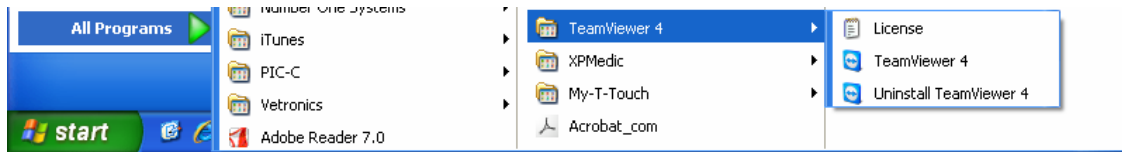

Once Teamviewer is running you will need to tell Technical Support the ID and Password so that they may connect to the PC.

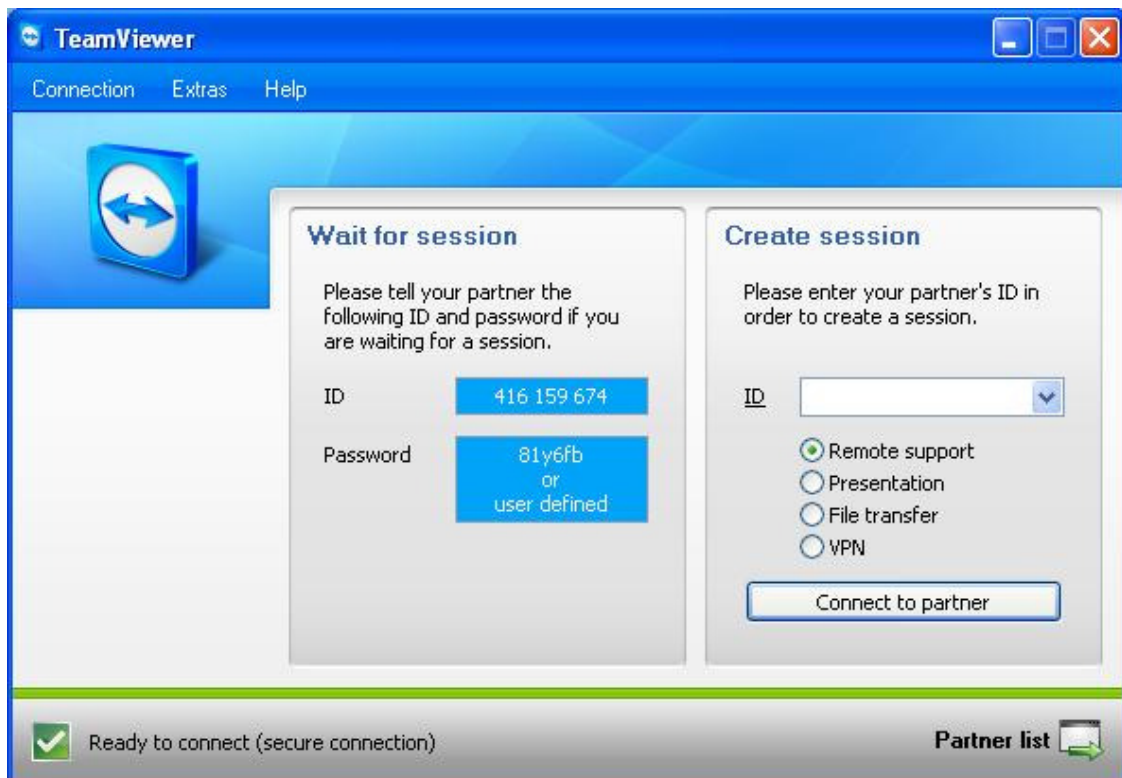

Once a connection is established, Technical Support will be able to view and control your PC in the same manner that you do in order to identify any problems or for program updates.