



Installing new PC software for Tafonius

The new software will either be available on the web site, via an e-mail or on a USB stick. It is necessary to use a USB stick to put the new software on Tafonius, so if the software has been downloaded or e-mailed to you, copy it to a USB stick – note it is not necessary to unzip the file that you have been sent.

Turn on the Tafonius PC either by turning the whole machine on, or by using the on/off button on the side of the PC. The software will automatically load as normal and stop at the Tafonius Shell, where you have the buttons for “Tafonius”, “View Data” etc.

At this point use the keypad and press alt-F4 to close the Tafonius shell and reveal the normal Windows desktop.

Plug the USB stick into Tafonius: On new machines, this can be in to the two spare USB slots on the Solomon monitor or into a spare slot on the PC. When the USB stick is inserted you may need to wait while the drivers for that device are loaded. Once the drivers are loaded or Windows recognises the USB stick, use **Start --> All Programs --> My Computer**, to find the USB stick and the zip file. Double-touch the zip file to show the contents of the zip file which can be one or 2 setup files. Double-touch either of these to start the installation. Follow the installation, accepting the default settings to install the new software.

When the installation is complete, double-touch the Tafonius Shell icon on the desktop to run the new software.

(or use **Start --> All Programs --> Vetronic Services Limited --> Tafonius**, and then the latest version).

When the Tafonius software starts, run through the Initialisation routine as normal and then when the normal Tafonius monitoring screen is showing, do the following:

Use the keyboard to enter **alt-z** and bring up a Customs Settings box. The last tab on the right is labelled “Gas Data”. Select that tab and then in the Data Type group box select your gas monitor.

If your monitor is Artema (sidestream, with a water trap) make sure the Artema enabled box is ticked.

If your monitor is IRMA (blue mainstream sensor) select IRMA, but do NOT select “VitalStore Compatibility” unless asked to do so. Click OK to close the Settings dialog box.

Installation is complete.