



## Retrieving Tafonius Log Data

**Scope:** This document relates only to full Tafonius machines when running the Tafonius software. Log data is not available for Stripped Tafonius or Junior machines

### General

There are a number of files saved with each patient anaesthetic session. If the patient details are not entered then the patient name details will be automatically saved as "Anonymous" and the data will be saved in a directory based on the date and time of the start of the session. In the picture of a selection dialog box below, the patients have not been named, so the highlighted session is for an un-named patient on February 25th 2009.

### Exporting the Slow Data Log


To retrieve the Slow Data Log you will need to export it using a button from the Tafonius shell. The Tafonius shell is the white screen with the Tafonius button and View Data, Export Data buttons.





### **Export Data**

Insert a USB stick in the Solomon USB ports or into a spare USB port on the PC. From the Tafonius shell, click on the Export Data button which opens a browser to allow you to choose the data to export. All data in the browser is listed in date order with the most recent files at the top of the browser.

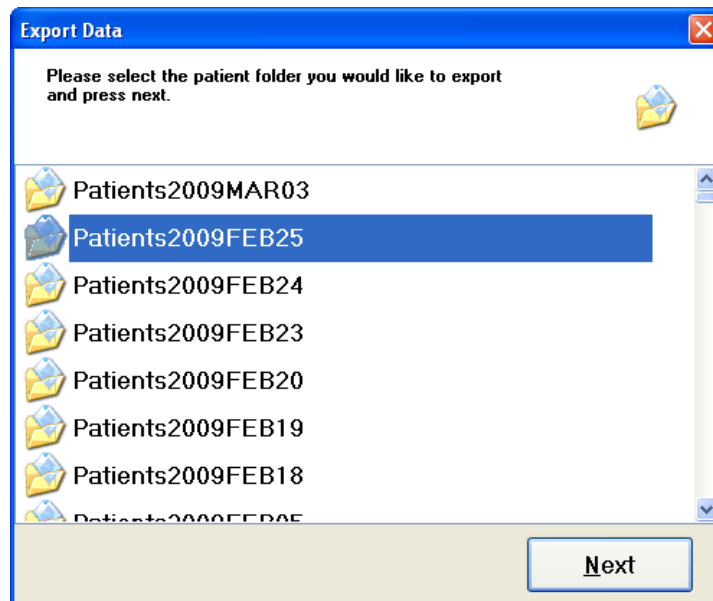

First select the patient that you wish to export data for and then click or touch the Next button. Then choose your destination drive (usually a USB flash drive named D:\) and touch or click the Export button. The Slow Data and the Anaesthetic Record data will be copied to your chosen destination. Send these files to Vetronic Services Limited via an email.

If you are requested to send the Windows Event Log, proceed as follows:

### **Exporting the Windows Event Log**

- Exit Tafonius
- From the Start menu click Control Panel->Administrative Tools->Event Viewer. (You can access the start menu by clicking on the windows key on the keyboard. )
- In the left-hand panel under “Event Viewer (Local)” Click on “Application” so that it becomes highlighted
- Click on the “Action” menu option and “Save Log File As”.
- If you've got a memory stick plug this in to one of the free USB ports on the ELO, give the file a name and then save the file to a location on the stick.

Please send this file to Vetronic Services Limited via an email.