





Leak Testing in Auxiliary mode for Junior and Tafonius machines



Start the machine in Auxiliary mode. Do not go into the Windows program. Press and hold one of the Auxiliary control knobs when the Auxiliary screen says, "PUSH ANY KNOB FOR SYSTEM LEAK TEST". If you miss the prompt, you will need to restart the machine to be prompted again.

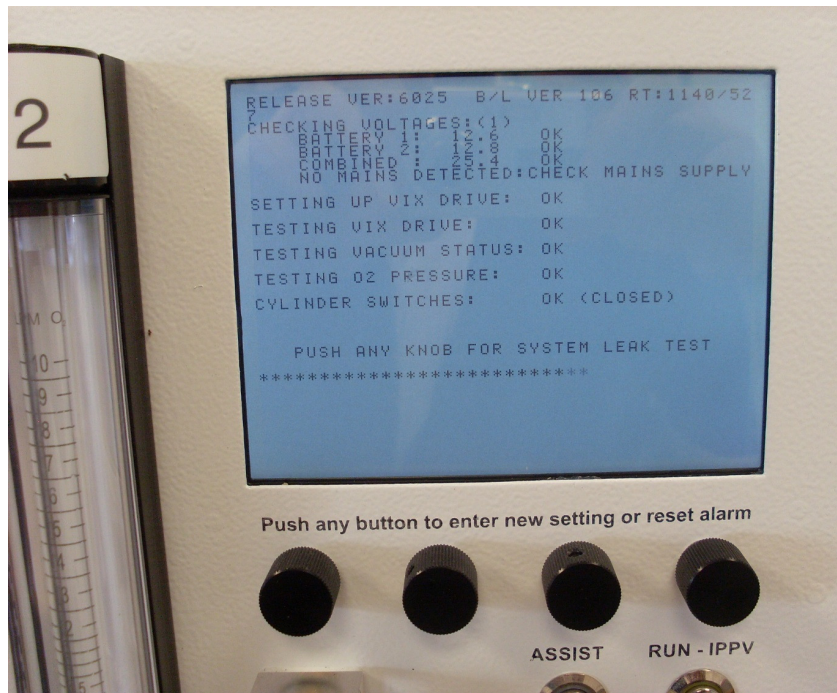


Once in this Auxiliary leak test mode, remain here for all of the following tests

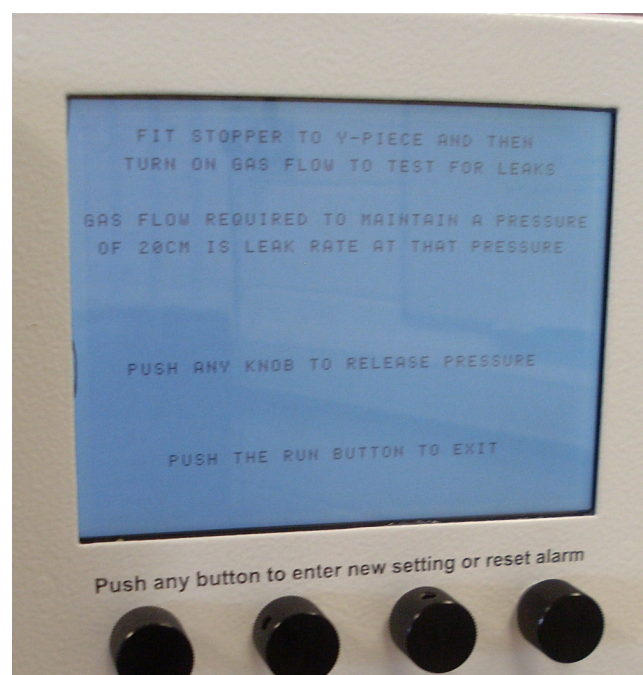

Run the manual Leak Test

Add fresh gas to the patient circuit using the oxygen flow meter to increase the pressure, as shown by the manometer, to 20cm H₂O.

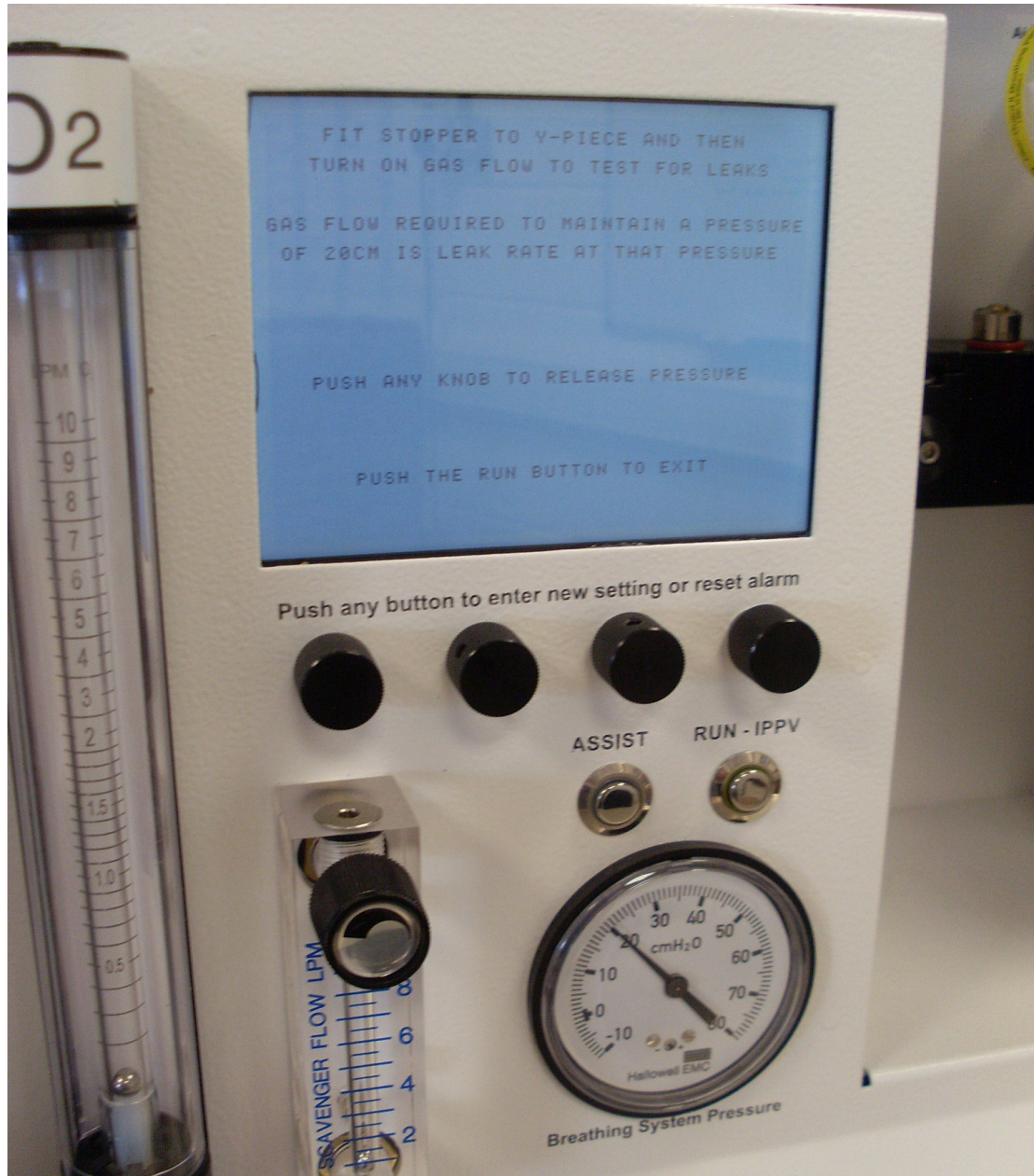

When the dial shows 20cm pressure, turn off the flow meter and observe the manometer. If the pressure starts to fall, add gas using the oxygen flow meter until the pressure is maintained at a steady 20cm pressure. Note the flow rate on the oxygen flow meter. This is the leak value of your machine at 20cm pressure. If the value is in excess of 500mls use the user guides to isolate and repair the leak.